


Raised Toilet Seat by


WARNINGS

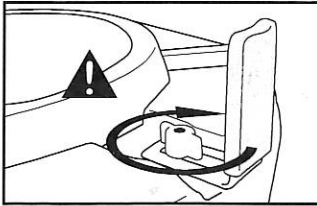

Never overtighten the clamps as this could cause damage to the toilet bowl or the seat itself.



Check your raised toilet seat every week to ensure **all** clamps are securely tightened and snug against the toilet bowl.

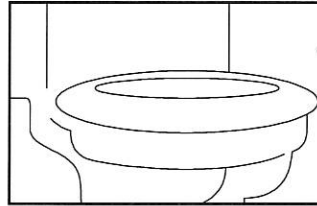

For use only on standard toilets. Seat dimensions: 15.75" x 15.25" (40 cm x 38.7 cm)




DO NOT exceed weight capacity!



DO NOT USE THE RAISED TOILET SEAT IF THERE ARE ANY CRACKS OR DAMAGED COMPONENTS!

CARE AND USE

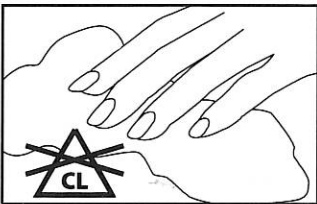


DO NOT use chlorine bleach or abrasive cleansers. To clean, wipe down with a damp cloth using a mild detergent only.




Supervision or assistance when using seats is recommended for users with limited mobility.




Before installing your raised toilet seat, make sure your toilet is dry and dirt free.

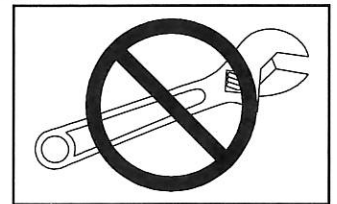


No tools are required.

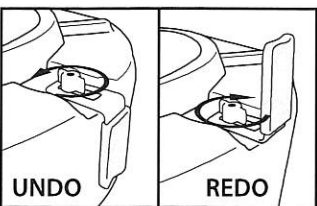


The clamps may be inverted for shipping. If so, undo the clamps and reposition them as shown.

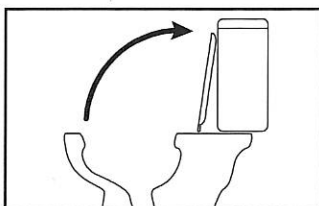

Lift the existing seat/lid or remove entirely if the raised toilet seat will be left in place.

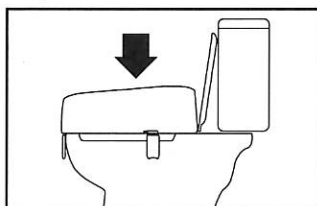

Position the raised toilet seat onto the toilet. Ensure that the seat rests well on the toilet bowl.

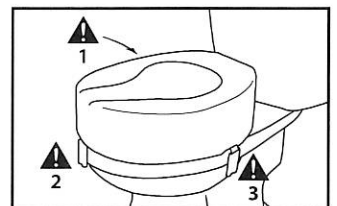

Adjust the three clamps so that they are snug against the toilet bowl.



READ ALL WARNINGS CAREFULLY. KEEP THIS INFORMATION FOR FUTURE REFERENCE.

If you have any problems or require assistance with this product please contact:

Parsons ADL Inc.
1986 Sideroad 15, Tottenham, ON
Canada L0G 1W0

1-800-263-1281
www.parsonsadl.com

MADE AND PRINTED IN THE UK



WARRANTY: This warranty is extended to the original purchaser of this product for a period of one year from the original purchase date against defects in materials and/or workmanship and is dependent that the product has been used in the intended fashion as outlined on this sheet. Rubber components are not warranted. Any installation, service and/or labour charges are at the discretion of the vendor and are not the responsibility of Parsons ADL. Any parts damaged by improper installation, repair and/or alteration by an unauthorized agency or individual, or parts damaged by neglect, accident or misuse on the part of the agency or individual are not warranted. Parsons ADL's sole liability extends to repair or replacement of parts or components as previously specified. The foregoing is in lieu of any other warranties expressed or implied, whether written or oral.