EYE DROP GUIDE

Holds the eye open and directs the drops, allowing for accurate eyedrop installation! Clips into place over most eye drop bottles and positions the bottle at the correct angle over the eye.

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www.parsonsadl.com

16M002A

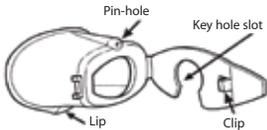
0 65292 27002 9

Parsons ADL Inc.

Tottenham, ON Canada L0G 1W0

Special cup prevents blinking by keeping the lower eyelid open. Unique pinhole directs eyesight upward and away from the descending drops.

INSTRUCTIONS:

1. Open Autodrop®. Loosen cap from bottle but do not remove completely.
2. Place the keyhole slot around the base of the bottle neck. Remove cap.
3. Close the Autodrop®. Do not force the bottle nose into Autodrop®, it should close easily.
4. Hold down lower eyelid. Place Autodrop® over the eye with the lip of the Autodrop® against the cheek.
5. Tilt head backwards, look through pin-hole towards the light.
6. Squeeze bottle gently to allow prescribed drops to fall

into the eye.

7. Open Autodrop® and replace top on eye drop bottle.

8. To clean Autodrop® remove eye drop bottle and wash Autodrop® in warm soapy water. Rinse and dry thoroughly.

The manufacturer recommends that Autodrop® is used for the whole course of treatment for an eye disorder. When an eye drop bottle is discarded remove Autodrop® and clean thoroughly before fitting to the new bottle. Autodrop® may be used on more than one type of eye drop at the same time but if you suffer from more than one eye complaint we recommend that you use a separate Autodrop® for each type of eye drop.

For clinics and hospitals: Autodrop® may be autoclaved on a low pressure cycle for 30 minutes. Release clip to allow moisture to evaporate. Autodrop® can also be cleaned in sterilizing fluid. Rinse and dry thoroughly.

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