
Clear Transparent WHEELCHAIR TRAY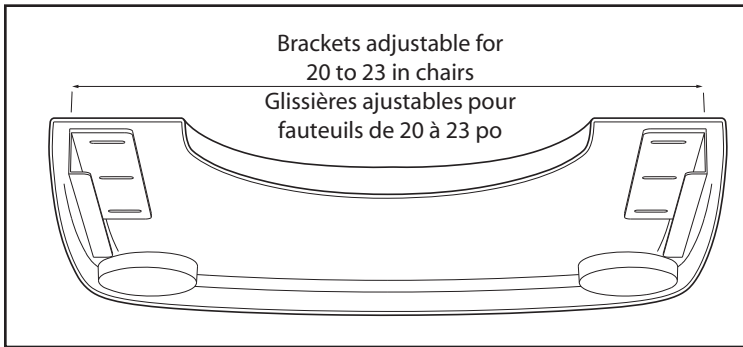

TRAY SIZE: 24 x 21 in (61 x 53 cm)

Dimensions de la tablette : 24 x 21 po (61 x 53 cm)

WARNING - AVERTISSEMENT

Note: Check all parts for shipping damages before using. In case of damage, do NOT use this tray. Contact the Dealer for further instructions.

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This molded transparent tray with BLIND FASTENING SYSTEM has all smooth edges so there is no place for dirt or germs to collect. Easy to wipe clean. Rounded corners and edges protect tender skin. There are two recessed wells at the front corners for drinking cups, etc. The tray is raised slightly around the torso cutout to help reduce minor spills from reaching the user's lap. The transparent material makes it easy for the person to see the front corners of the chair through the tray while the chair is in motion. A great safety feature. Lightweight and strong.

INSTRUCTIONS: The tray mounts firmly to the wheelchair by sliding onto the armrests, using our adjustable slide on bracket system. This tray will mount on flat, padded or tubular armrests.

For proper fit: Adjust brackets so that the inside edge of the brackets matches the outside to outside arm width of the wheelchair that the tray is to be placed on.

CARE: This clear tray is composed of an extremely durable PETG plastic resin that allows surface scratches to be removed with a heat gun or hair dryer. To clean, wipe with a damp cloth. You may use detergent soap or anti-bacterial disinfectants.

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