

# OPERATING MANUAL

## ASSEMBLY/USAGE of LOOK SERIES RAISED TOILET SEATS

FOR MODELS:

1662 - 2 in (5 cm)

1663 - 2 in (5 cm) with LID


1664 - 4 in (10 cm)

1665 - 4 in (10 cm) with LID


### WARNINGS



**Never** overtighten the clamps as this could cause damage to the toilet bowl or the seat itself.

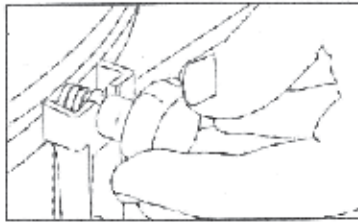

**Check** your raised toilet seat every week to ensure all clamps are securely tightened.

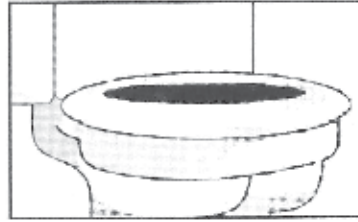


**Only** used the raised toilet seat on toilets that meet the specifications. Ensure the seat rests well on the toilet frame.




**DO NOT** exceed weight capacity!

**! DO NOT USE THE RAISED TOILET SEAT IF THERE ARE ANY CRACKS OR DAMAGED COMPONENTS!**

### CARE AND USE

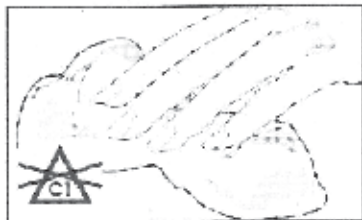

**DO NOT** use chlorine bleach or abrasive cleansers. To clean, wipe down with a damp cloth using a mild detergent only.

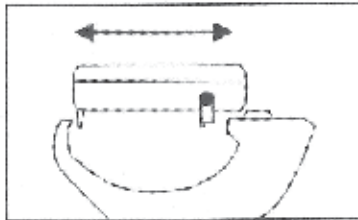

This raised toilet seat will only fit standard toilet seats (approximate size: 12 $\frac{3}{4}$ " x 10 $\frac{1}{4}$ "/32cm x 26cm).



**Supervision** or assistance when using seats is recommended for users with limited mobility.


**READ ALL WARNINGS CAREFULLY**

**KEEP THIS INFORMATION FOR FUTURE REFERENCE**


### ASSEMBLY




Before installing your raised toilet seat, make sure your toilet is dry and dirt free.

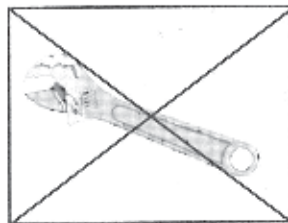


No tools are required.

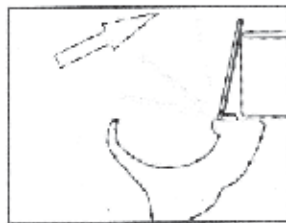

Lift the existing seat and place the raised toilet seat onto the toilet bowl.

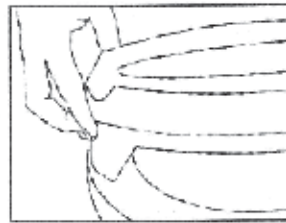

Position the front lip against the front of the toilet bowl.

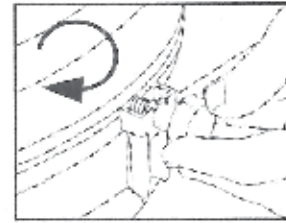

Turn both adjustment knobs clockwise until the rubber buffers touch the toilet bowl.

### WARRANTY

This warranty is extended to the original purchaser of this product for a period of one year from the original purchase date against defects in materials and/or workmanship and is dependent that the product has been used in the intended fashion as outlined on this sheet. Rubber components are not warranted. Any installation, service and/or labour charges are at the discretion of the vendor and are not the responsibility of Parsons ADL. Any parts damaged by improper installation, repair and/or alteration by an unauthorized agency or individual, or parts damaged by neglect, accident or misuse on the part of the agency or individual are not warranted. Parsons ADL's sole liability extends to repair or replacement of parts or components as previously specified. The foregoing is in lieu of any other warranties expressed or implied, whether written or oral.

**IF YOU HAVE ANY PROBLEMS OR REQUIRE ASSISTANCE WITH THIS PRODUCT PLEASE CALL:**

**800.263.1281**



PARSONS ADL INC.  
1986 Sideroad 15, Tottenham, ON L0G 1A0

Accessories for Daily Living