

20H050 - EZ ENABLER TABLE



Step 1: Remove 2 screws from clamp using on of the Allen keys.



Step 2: Position the clamp on the wheelchair tubing to achieve desired placement and tighten the 2 screws to secure the clamp around the tubing with the Allen key.



Step 3 & 4: Using the 2 smallest Allen keys position the end of the clamp into which the upright aluminum tubes will fit. Step 3 adjusts up and Step 4 adjusts left or right.



Step 5: Insert the longest upright tube in the clamp. Stretch one of the rubber rings over the tube to the position where you want the cup holder to sit. The rubber ring will prevent the holder from sliding down on the tube.



Step 6: Add one or two of the tubes to add to the height. Position the cup holder on the tube.



Step 7: Place the other rubber ring on the tube where you want to place the hook. Note: the small screw and washer included is used to tighten the hook in place, although the rubber ring will also serve this purpose.



Step 8: Place the tray on the tube. The tray will swivel as needed.

www.parsonsadl.com
Parsons ADL Inc.
1986 Sideroad 15, Tottenham L0G 1W0
800.263.1281

