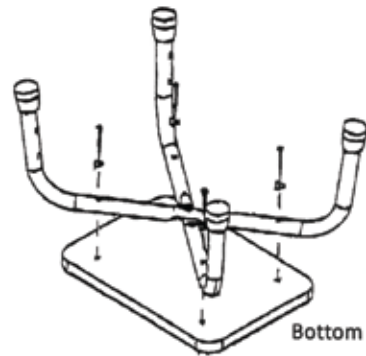
STEP STOOL WITH HANDRAIL 16K055A

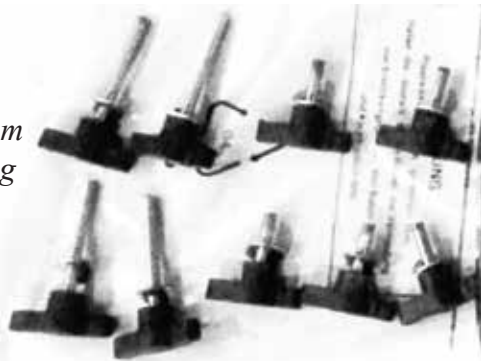
Please inspect all parts carefully for cracks/breaks prior to use.

ASSEMBLY INSTRUCTIONS

1. Turn the stool portion upside down and place it flat on a table. Then place the leg frame crosswise and upside down on the stool top (the holes in the leg frames must be lined up with the holes in the stool top.) Use 5 pieces M6x20mm short wing screws and 5 washers to join the leg frames to the seat.

4 of M6x50mm long wing screws


5 of M6x20mm short wing screws

2. Attach the handrail to the leg frame using 4 pieces M6x50mm long wing screws and 4 washers.

Made in China

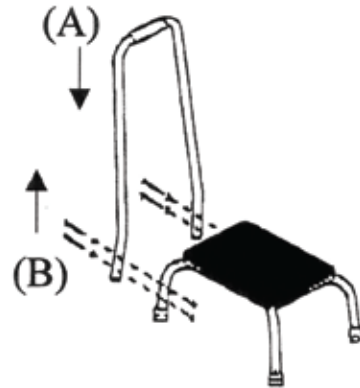3. Insert top rail (A) into bottom (B) rail until the push button pops out in place. Be sure rail is angled away from the stool.

USER INSTRUCTIONS

This product is intended to be used as a step stool. The handrail is provided to aid in balance.

DO NOT put full weight onto the handrail. This can cause stool to tip over and cause injury to user.

CARE INSTRUCTIONS

Periodically check that all screws are tight. Clean with mild detergent and water. Dry thoroughly to avoid rust.

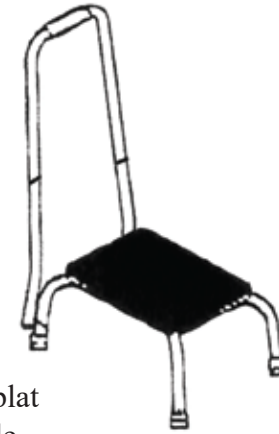
Hardware included:

- 5 of M6x20mm wing screw
- 4 of M6x50mm wing screw
- 9 of washers

Persons with severe disabilities or persons who are unable to walk without an aid should not attempt to use this product without assistance.

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Parsons ADL Inc.