


DELUXE RIGID SOCK AID - Enfile bas rigide de luxe

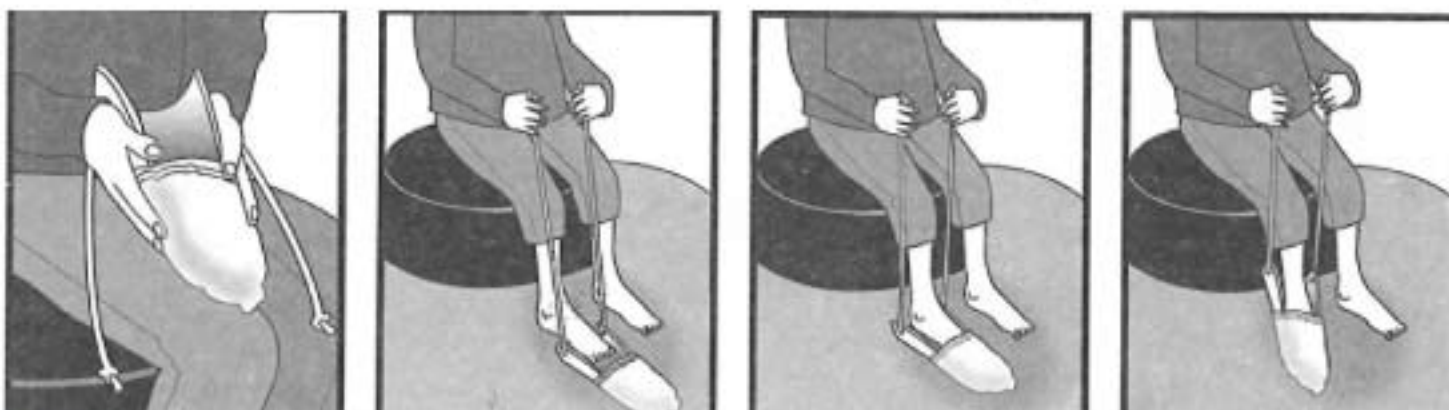

INSTRUCTIONS:

1. Gather stocking to the toe with your thumbs inside.
2. Pull stocking onto plastic blade.
3. The stocking must be pulled on tightly to the toe checking that the heel is underneath and not twisted to one side.
4. Sitting securely on your bed or chair, lower the stocking aid to the floor while holding the cords in your hands.
5. Slide your foot into the plastic blade and wiggle your foot into the toe of the stocking.
6. At this point lift heel off the floor, straighten leg and point toes. Pull cords alternately with a quick wiggling action and stocking will be pulled up the leg until you can reach it.

DIRECTIVES:

1. Ramassez le bas jusqu'à la pointe du bas, avec vos deux pouces à l'intérieur.
2. Étirez le bas et le mettre sur la partie de l'enfile bas sans cordons.
3. Le bas doit être bien tiré jusqu'à la pointe du bas. Vérifiez que le talon du bas est placé vers le sol et qu'il n'est pas entortillé.
4. Assoyez-vous confortablement sur le lit ou une chaise, baissez la lame avec le bas jusqu'au sol pendant que vous tenez les cordons dans vos mains.
5. Glissez votre pied dans l'enfile bas et agitez votre pied vers la pointe du bas.
6. Ensuite, levez le talon, redressez la jambe et pointez les orteils. Tirez les cordons vers vous tout en agitant votre jambe. Une fois le bas suffisamment haut, vous pourrez retirer la lame et finir de remonter le bas.



16D037A